



Sway Medical


Baseline Testing

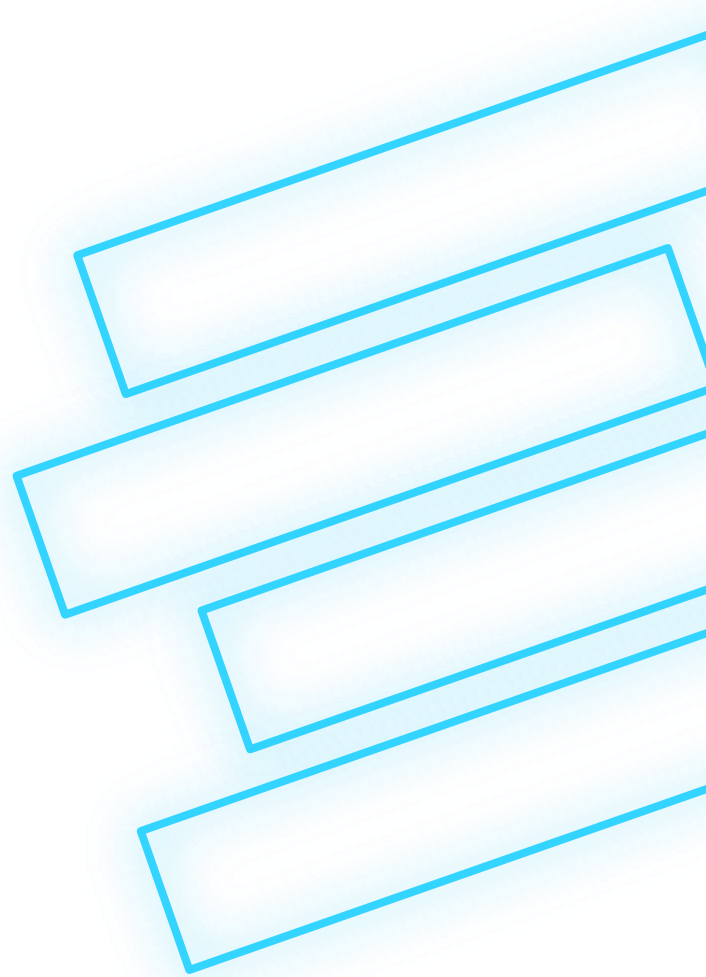



Scan QR code to download
the Sway Medical App

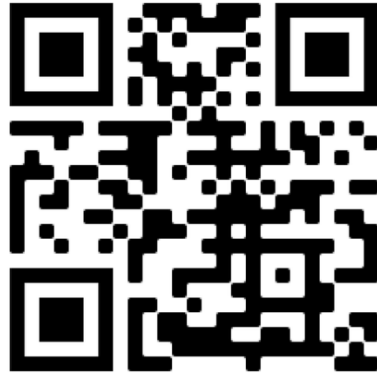

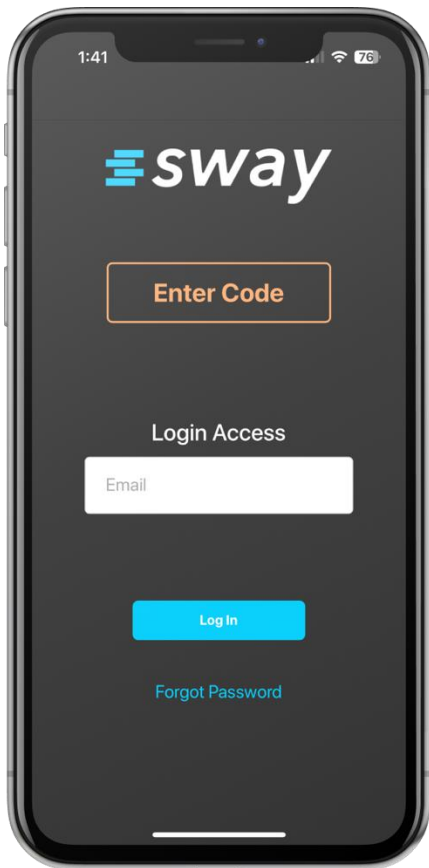

- Put your phone on **DO NOT DISTURB**
- Close out all other apps
- Disable power mode or rise to wake
- Enable rotate screen
- Phone battery more than 10%



1

Enter
code


2

Create a
profile

3


Complete balance +
cognitive tests




 Symptoms

 mBESS

 Inspection Time

 Reaction Time

 Impulse Control

 Memory






Enter the 5-digit
code at the TOP
of your screen



AGES:9-11[FPQCD]

AGES:12+[ABUUM]

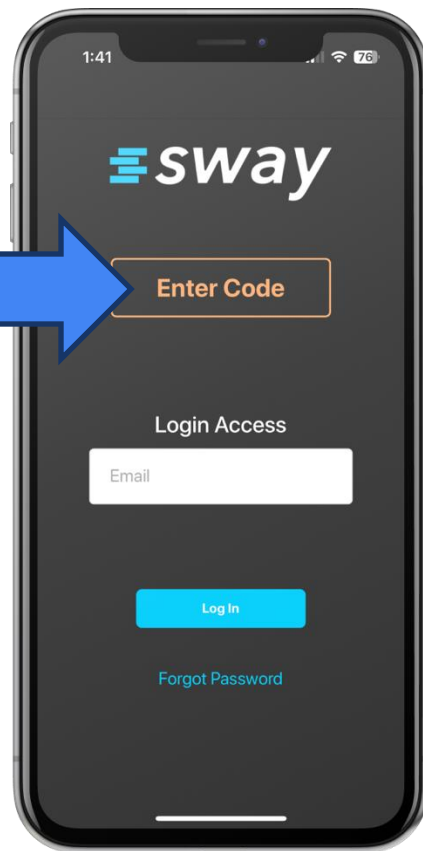



Create a Profile

- Basic information
- Education
- Medical

Click “Finish” once all fields are filled out.


The image shows a smartphone screen displaying the 'Manage Profile' app. At the top, the status bar shows the time 10:37 and signal indicators. The app header has a 'Cancel' button and the title 'Manage Profile'. Below the header is a blue icon of a person holding a heart. The main text reads 'Nice to meet you, we're Sway!' followed by a subtext: 'We look forward to getting to know you. Let's get your profile set up so we can get started.' Below this is a progress indicator with four dots, the first of which is blue. The 'Profile Information' section contains two text input fields: 'First Name' with the placeholder 'ex. John' and 'Last Name' with the placeholder 'ex. Smith'. At the bottom of this section is a blue button labeled 'NEXT: BASIC INFORMATION'.



Click Hardware Verification!

Then start the symptoms checklist

- Complete the symptoms based on how you feel TODAY

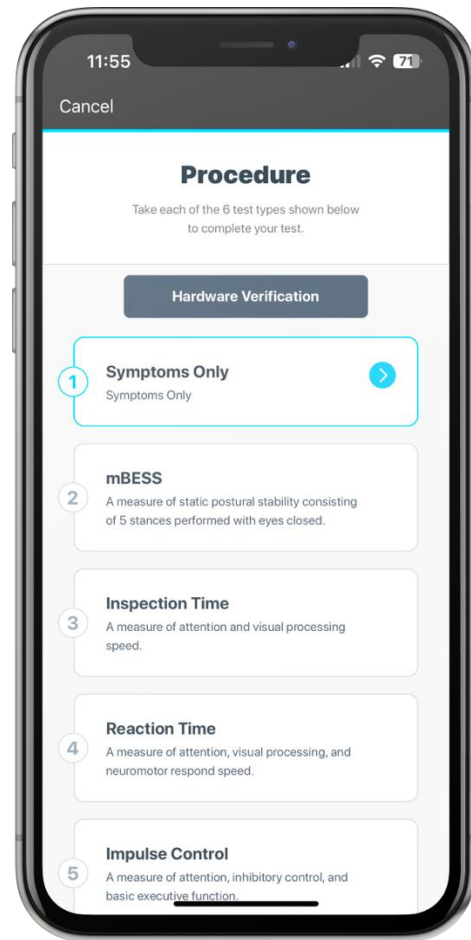



Follow the instructions on the screen!

- For each test you will complete a practice trial then 2-3 actual trials
- After you complete the balance test – sit down to complete the rest
- Check in with me before you leave the testing site

